

CATALOGUE

UNITED INTERIM SDN BHD

17B, Jalan Kota Raja F 27/F, Hicom Tower Centre,
Seksyen 27, 40400 Shah Alam, Selangor Darul Ehsan.

 www.uisbgroup.com

 info@uisbgroup.com

 603-51929004

 6012-4673347

CORPORATE OVERVIEW

At **United Interim Sdn Bhd**, we believe that healthcare providers deserve a dependable partner who understands the importance of quality, safety, and efficiency. Since our incorporation on 19 January 2011, and with our headquarters located in Shah Alam, we have proudly built our reputation as a trusted supplier of premium medical products across Malaysia.

With a strong emphasis on affordability without compromising on quality, our carefully curated range of medical products is designed to meet the evolving needs of hospitals, clinics, and healthcare facilities. Our experienced team and responsive customer service ensure reliable, consistent value in every partnership.

More than just a supplier, United Interim Sdn Bhd is your dedicated healthcare partner focused on helping you deliver exceptional care. Through continuous improvement, technical expertise, and a client first approach, we grow alongside the professionals and communities we serve.



Vision

Our commitment is to provide innovative and forward thinking solutions through our well established and integrated network, leveraging expertise, collaboration, and advanced resources to deliver exceptional value and sustainable growth for our stakeholders.



Mission

To be a leading name in the healthcare supply industry by delivering superior, high quality services, driving innovation, and setting new standards of excellence in everything we do.



Objective

To ensure all planning and operational activities are executed in alignment with established guidelines, consistently reflecting the company's defined vision and mission.

SINGLE & COMBO DRUG TEST



BENEFITS

- Simple and rapid
- No special equipment is needed
- Results are easy to interpret
- No sample preparation
- Results in 5 minutes

QUALITY CONTROL

A procedural control is included in the test. A line appearing in the control region (C) is considered an internal procedural control. It confirms sufficient specimen volume, adequate membrane wicking and correct procedural technique. Control standards are not supplied with this kit. However, it is recommended that positive and negative controls be tested as good laboratory practice to confirm the test procedure and to verify proper test performance.



SUMMARY AND EXPLANATION OF THE TEST

FastDetect™ Drug Rapid Test is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes monoclonal antibodies to selectively detect elevated levels of specific drugs in urine.

TEST PROCEDURE

Allow the test, urine specimen, and/or controls to reach room temperature (15–30°C) prior to testing.

1. Bring the pouch to room temperature before opening it. Remove the Test Dipstick from the sealed pouch and use it within one hour.
2. With arrows pointing toward the urine specimen, immerse the test dipstick vertically in the urine specimen for at least 10–15 seconds. Do not pass the maximum line (MAX) on the Test Dipstick when immersing the strip. See the illustration below.
3. Place the Test Dipstick on a non-absorbent flat surface, start the timer and wait for the colored line(s) to appear. Read results at 5 minutes. Do not interpret the result after 10 minutes.

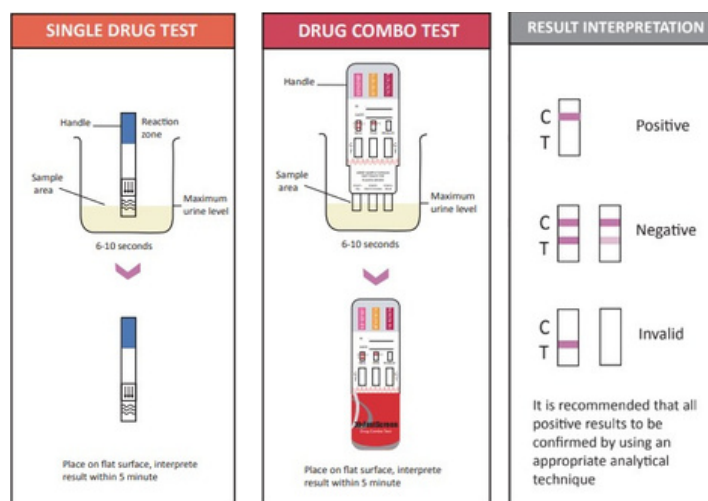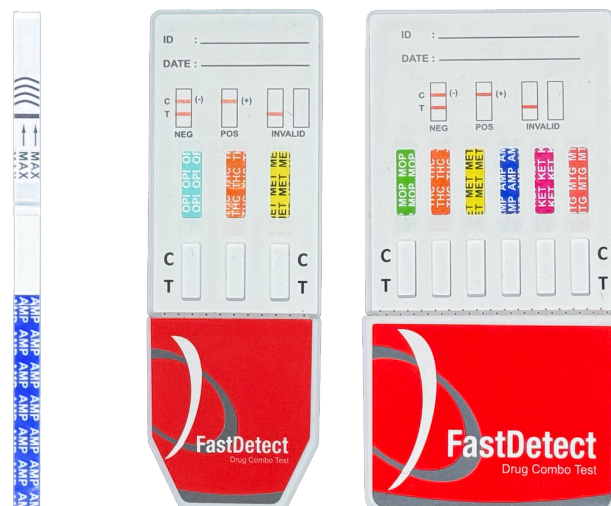

INTERPRETATION OF RESULTS

(Please refer to the illustration above)

NEGATIVE:

Two lines appear. One colored line should be in the control line region (C), and another apparent colored line should be in the test line region (T). This negative result indicates that the concentration is below the detectable level.

NOTE: The shade of color in the test line region (T) may vary, but it should be considered negative whenever there is even a faint colored line.

POSITIVE:

One colored line appears in the control line region (C). No line appears in the test line region (T). This positive result indicates that the concentration exceeds the detectable level.

INVALID:

Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test using a new test. If the problem persists, discontinue using the lot immediately and contact your local distributor.

YELLOW & BLUE CONTAINER

Blue Bottle Features



Specification	Details
Product Code	LAB-13
Product	2 Start Thread
Dimension	55mm (D) x 732mm (H)
Weight	19gm
Material	Polypropylene
Leakage Airtightness	Leak proof quality
Packaging	500 Pcs/Carton

Yellow Bottle Features

Specification	Details
Product Code	LAB - 06
Product	3 Start Thread
Dimension	44mm (D) x 56mm (H)
Weight	19gm
Material	Polypropylene
Leakage Airtightness	Leak proof quality
Packaging	500 Pcs/Carton



FLIP TOP CONTAINER

Usage/Application

- Urine specimen collection for diagnostic testing
- Workplace or pre-employment drug testing
- Hospitals, clinics, and laboratory sample storage
- Research and clinical trial sample collection
- Educational and training purpose for demonstration, training programs and procedural practice

Specimen Storage Urine

Specimens should be stored at 2°C to 8°C for a maximum of 48 hours prior to testing. For extended preservation, specimens may be frozen at temperatures below -20°C. Prior to testing, frozen specimens must be thawed completely and mixed thoroughly to ensure sample integrity.

Product Advantages

- Hygienic & Safe: Flip-top cap reduces risk of contamination.
- Secure Transport: Airtight seal prevents leaks during storage and handling.
- Lab Ready: Compatible with most diagnostic workflows.



Key Features

- Built-in Temperature Sticker: Instantly shows urine temperature to detect fake or tampered samples.
- Leak-Proof Flip Cap: Secure and easy to open/close with one hand.
- Durable & Flexible: Made for tough use in clinical or workplace environments.
- Volume Markings: Graduated for accurate volume measurement.

Product Description

- Product Code: LAB - 11
- Dimension: 48mm (D) x 60mm (H)
- Weight: 19gm
- Material: Polypropylene
- Mold Code: WP-69
- Visibility: Excellent, fresh clear and can see through (to differentiate urine color or fake urine)

Compliance & Safety Certifications

- ISO 9001:2015 Certified Manufacturing
- Complies with Clinical & Laboratory Standards Institute (CLSI) guidelines
- Non-toxic, BPA-free, and medical-grade material



URINE SPECIMEN TRANSPORT BAG & TEMPERED VOID SECURITY LABEL

Bag Features

Specification	Details
Material	Featuring a dual layer design with water resistant nylon and waterproof PVC for maximum performance
Insulation	Foam padded inner lining
Capacity	Up to 12 containers (standard)
Closure System	Heavy-duty zipper came with zip-tie
Strap	Adjustable, detachable shoulder strap
Colours Variation	Black & Blue



Sticker Features

Specification	Details
Tamper-Evident Function	Displays a visible VOID watermark or residue when sticker is peeled off
Material	Temper-evident security film
Applications	Forensic sample logging Health screenings
Function	To seal and identify urine or biological specimen containers
Durability	Water-resistant & chemical-resistant
Colours Variation	Bright Blue
Code	Customable sequence code number



ORDERING INFORMATION

CATALOG NO.	PRODUCT NAME	SENSITIVITY (NG/ML)	PACKAGING
SINGLE STRIP			
FD-UI01-BZO	FastDetect Drug of Abuse Rapid Test, Benzodiazepines	300	100 T/box
FD-UI01-KET	FastDetect Drug of Abuse Rapid Test, Ketamine	1000/500	100 T/box
FD-UI01-MTG	FastDetect Drug of Abuse Rapid Test, Mitragnine	300/500	100 T/box
FD-UI01-MET	FastDetect Drug of Abuse Rapid Test, Methamphetamine	1000	100 T/box
FD-UI01-MOP	FastDetect Drug of Abuse Rapid Test, Morphine	300	100 T/box
FD-UI01-THC	FastDetect Drug of Abuse Rapid Test, Marijuana	50	100 T/box
FD-UI01-AMP	FastDetect Drug of Abuse Rapid Test, Amphetamine	1000	100 T/box
FD-UI01-BAR	FastDetect Drug of Abuse Rapid Test, Barbiturates	200/300	100 T/box
FD-UI01-COC	FastDetect Drug of Abuse Rapid Test, Cocaine	300	100 T/box
FD-UI01-ER5	FastDetect Drug of Abuse Rapid Test, Erimin	500/1000	100 T/box
FD-UI01-MDMA	FastDetect Drug of Abuse Rapid Test, Ecstasy	500/1000	100 T/box
FD-UI01-MTD	FastDetect Drug of Abuse Rapid Test, Methadone	200/300	100 T/box
FD-UI01-OPI	FastDetect Drug of Abuse Rapid Test, Opiate	300	100 T/box
DRUG COMBO			
FD-UI02	FastDetect Drug Combo Test 2-in-1 Dipcard	-	25 T/box
FD-UI03	FastDetect Drug Combo Test 3-in-1 Dipcard	-	25 T/box
FD-UI04	FastDetect Drug Combo Test 4-in-1 Dipcard	-	25 T/box
FD-UI05	FastDetect Drug Combo Test 5-in-1 Dipcard	-	25 T/box
FD-UI06	FastDetect Drug Combo Test 6-in-1 Dipcard	-	25 T/box
DRUG TWIN CARD			
FD-UI07	FastDetect Drug Combo Test 7-in-1 Dipcard	-	25 T/box
FD-UI08	FastDetect Drug Combo Test 8-in-1 Dipcard	-	25 T/box
FD-UI09	FastDetect Drug Combo Test 9-in-1 Dipcard	-	25 T/box
FD-UI10	FastDetect Drug Combo Test 10-in-1 Dipcard	-	25 T/box
FD-UI11	FastDetect Drug Combo Test 11-in-1 Dipcard	-	25 T/box
FD-UI12	FastDetect Drug Combo Test 12-in-1 Dipcard	-	25 T/box
YELLOW, BLUE AND FLIP CONTAINER			
LAB-11	Flip Top 60ml Urine Container	-	200 P/ctn
LAB-12	60ml Yellow Cap Urine Container	-	500 P/ctn
LAB-13	100ml PP Specimen Container, Blue Cap	-	500 P/ctn



www.uisbgroup.com

UNITED INTERIM SDN BHD (930360-K)



United Interim Sdn Bhd



uisbgroup



united.interim



uisbstore



uisbstore